

How to nebulise with the VELOX®



1

Wash your hands thoroughly before each inhalation session.



2

If you are running the device using batteries (or rechargeable batteries), insert them before assembling the nebuliser.



3

Place the medication reservoir flush with the aerosol chamber and close the locking tabs.



4

Insert the nebuliser into the controller (both areas must be completely dry). Close the tab on the underside of the device.



5

Fit the mouthpiece or mask onto the nebuliser.



6

Unscrew the medication cap. Fill with medication (at least 2 ml, not more than 6 ml). Screw the medication cap onto the nebuliser.



7

If you are operating the device with the power adapter, connect the controller to the power adapter and a suitable socket.



8

Sit upright and in a relaxed position. Switch on the device. The device emits an acoustic signal, the LED on the button lights up green. Aerosol collects in the nebuliser.



9a

Hold the VELOX horizontally. Take the mouthpiece between your teeth and close your lips around it.



9b

If using a mask, hold the VELOX horizontally. The mask must fit snugly.



10

Breathe in and out slowly and deeply, inhaling the aerosol that has collected in the nebuliser. Inhalation can be stopped as soon as no more aerosol comes out of the valve.



11

Switch off the device. The device emits an acoustic signal, the LED on the button switches off. For technical reasons a residual amount of medication remains in the nebuliser.



12

Operation with the power adapter: For complete disconnection of the device from the mains, disconnect the power plug.

Device signals

Switching on the device:	LED lights up green.	1 beep.
During operation:	LED shines green.	
Low battery:	LED flashes green.	
Battery flat/voltage too low:	LED flashes orange three times.	1 beep. The device switches itself off automatically.
The nebuliser is not connected to the controller:	LED flashes orange three times.	The device switches itself off automatically.
The max. operating time of 15 min. has been exceeded:	LED flashes green three times.	1 beep. The device switches itself off automatically.
Switching off the device:	LED switches off.	1 beep.

Follow the Instructions For Use for the VELOX at all times, as they provide a detailed description. They are included with the device. If you lose these instructions, you can download a copy at www.pari.com or request a copy from PARI Medical Ltd on +44(0) 1932 341122, or email: infouk@pari.com

Hygiene and maintenance of the VELOX®

Make sure that areas where the nebuliser and controller connect are completely dry before using it again, otherwise the device will not create an aerosol.

If the inhalation session lasts longer than usual, the aerosol head may be blocked. Rinse it with VELOXcare.



1

Cleaning

When preparing for use for the first time and after every use:

- Place the nebuliser (disassembled into its individual parts), the mouthpiece and the mask in warm tap water with a little dishwashing liquid for about 5 min.
- Rinse the individual parts thoroughly under running water.



2a

Rinsing the aerosol head with VELOXcare

Once a week if the device is used daily.

Insert the medication reservoir in the VELOXcare with the opening facing downwards (blue valves are uppermost), fold it closed and close the locking tabs.



2b

- Connect the VELOXcare to the controller. Close the tab on the underside of the device.
- Fill with 2.5 ml isotonic saline solution or distilled water.



2c

- If you are operating the device with the power adapter, connect the controller to the power adapter and the socket.
- Switch on the VELOX. The device emits an acoustic signal, the LED on the button lights up green.
- The rinsing function is executed.



2d

- When the rinsing fluid has run through, switch off the device.
- The device emits an acoustic signal, the LED on the button switches off.
- Disconnect the VELOXcare from the controller.
- Take the medication reservoir out, rinse the medication reservoir and the VELOXcare thoroughly with tap water.



3

Disinfection

Disinfect once a week after cleaning

- In distilled, boiling water (for at least 5 min.)
- or
- In a disinfectant, e. g. for baby bottles (for at least 6 min.)



4

Drying

After every cleaning and/or disinfection, place the parts on a dry, clean, absorbent surface and allow to dry completely.



5

Storage

Keep the parts in a dry, dust-free place.



6

Replace the nebuliser (VELOX Year Pack):

1 x per year,
or after
365 applications and
52 disinfections.

